

Blue to the sky. Orange to the thigh.

How to use EpiPen® and EpiPen Jr® Auto-Injectors.

Remove the EpiPen® Auto-Injector from the carrier tube and follow these 2 simple steps:



- Grasp with orange tip pointing downward
- **Remove blue safety cap by pulling straight up**
– do not bend or twist



- Place the orange tip against the middle of the outer thigh
- **Swing and push the auto-injector firmly into the thigh until it “clicks”**
- Hold in place for 3 full seconds



Built-in needle protection

- After injection, the orange cover automatically extends to ensure the needle is never exposed.

After using EpiPen®, you must seek immediate medical attention or go to the emergency room. For the next 48 hours, you must stay close to a healthcare facility or be able to call 911.



Trusted for over 25 years.